

User Manual - Manual Homecare Nursing Bed with Commode, Side Tilting & Chair Position

1. Product Overview

The Manual Homecare Nursing Bed is designed for enhanced patient care at home or in healthcare settings.

It offers multiple functionalities including commode provision, side tilting, and chair position-all operated manually through a 3-crank system.

2. Specifications

- Frame Material: Cold-rolled mild steel, powder-coated
- Head/Foot Board: ABS, detachable
- Side Rails: Aluminum alloy, collapsible
- Mattress: 3-inch coir + foam with cotton cover
- Castor Wheels: 5-inch medical-grade with individual brakes
- Load Capacity: Patient weight 150-200 kg, max 260 kg
- Dimensions: 80" L x 37" W x 20" H



3. Included Accessories

- Coir + Foam Mattress
- Stainless Steel IV Stand (height-adjustable)
- ABS Activity Tray
- Hair Wash Basin
- Removable Bedpan
- Collapsible Guardrails



4. Functional Adjustments (Manual Operation)

A. Backrest Elevation (0° to 75°)

User Manual - Manual Homecare Nursing Bed with Commode, Side Tilting & Chair Position

- Use the right-side crank.
- Turn clockwise to raise, counterclockwise to lower.

B. Leg Section Elevation (0° to 45°)

- Use the middle crank.
- Turn clockwise to raise, counterclockwise to lower.

C. Side Tilting (0° to 45°)

- Use the left-side crank.
- Enables gentle lateral tilt for patient repositioning.

D. Chair Position Adjustment

- Coordinate backrest and leg elevation to form chair shape.
- Ideal for eating, reading, therapy, or rest.

5. Commode Usage

- Slide the integrated bedpan into the commode opening.
- After use, pull out and clean with mild disinfectant.
- Always reinsert securely.

6. Safety Guidelines

- Ensure castors are locked during patient use.
- Do not exceed weight limit.
- Confirm all cranks are turned gently and stopped when resistance is felt.
- Use guardrails at all times to prevent accidental falls.

User Manual - Manual Homecare Nursing Bed with Commode, Side Tilting & Chair Position

7. Cleaning & Maintenance

- Clean frame weekly with a damp cloth and mild soap.
- Disinfect mattress cover, tray, and bedpan regularly.
- Lubricate crank joints every 3 months.
- Check for loose screws or damaged parts periodically.

8. Warranty Information

- Crank System Warranty: 1 Year
- Covers mechanical defects and operational issues under normal use.
- Free repair or replacement within warranty terms.

9. Support & Contact

For service, parts, or installation help:

Phone/WhatsApp: +91-9929545206

Email: morecare.rehab@gmail.com

Website: www.morecarebeds.com